**Navodilo za uporabo VIVAMEL – ALGINATNA OBLOGA Z MEDICINSKIM KOSTANJEVIM MEDOM**

Pred prvo uporabo izdelka Vivamel – alginatna obloga z medicinskim kostanjevim medom natančno preberite navodilo. Navodilo shranite. Morda ga boste želeli ponovno prebrati.

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**Instructions for use VIVAMEL – ALGINATE DRESSING WITH MEDICAL CHESTNUT HONEY**

Before first use of Vivamel – alginatne dressing with chesnut honey read the instructions carefully. Keep the instructions for future use.

**WHAT IS VIVAMEL – ALGINATE DRESSING WITH MEDICAL CHESTNUT HONEY?**

Vivamel – alginatne dressing with chesnut honey is a sterile dressing in the basis of calcium alginate that is impregnated with medical chestnut honey entirely. The dressing is suitable for treating a wide spectrum of wounds in different healing phases. It is especially suited for treating chronic wounds, which heal badly or slowly. The dressing is highly absorbent and is therefore a perfect choice for treating wounds with medium to large amounts of secretion. Available in two sizes: 10 x 10 cm and 5 x 5 cm. The larger dressing (10 x 10 cm) contains 20 g whereas the smaller one (5 x 5 cm) 5 g of medical honey.

**Medical honey**

Chestnut medical honey has a high sugar content (up to 80%), low water content (up to 17%) and acidic pH. Due to its physical properties, it has a high osmotic activity, which creates an unfavourable environment for the growth of microorganisms, stimulates wound cleansing and promotes wound healing. Honey enables moist healing and forms a mechanical protection on the wound.

**Calcium alginate**

Alginatne dressing is made of calcium alginate fibre, which is a natural polymer, gained from brown seaweed. Calcium alginate has a high ability of excrete absorption. Upon contact with excretes, the dressing forms gel, which maintains optimal moist environment for wound healing. Calcium ions from the dressing exchange with sodium ions from the wound, providing an additional haemostatic effect.

**WHICH WOUNDS ARE SUITABLE FOR TREATMENT WITH VIVAMEL – ALGINATE DRESSING WITH MEDICAL CHESTNUT HONEY?**

- lower leg ulcers
- diabetic ulcers
- pressure ulcers
- infected wounds
- malodorous wounds
- surgical wounds
- traumatic wounds
- burns
- moist necrosis

**ADVANTAGES OF USING VIVAMEL – ALGINATE DRESSING WITH MEDICAL CHESTNUT HONEY AND ITS EFFECT:**

- enables moist wound healing
- lowers risk of wound infection
- reduces unpleasant odours
- stimulates necrotic tissue cleaning
- stimulates granulation and epithelisation
- promotes wound healing and prevents scarring
- no sticking onto the wound
- does not cause pain at dressing change
- easy to use
- no damage on the area surrounding the wound

**HOW TO USE VIVAMEL – ALGINATE DRESSING WITH MEDICAL CHESTNUT HONEY?**

1. First, rinse the wound with lukewarm running drinking water or saline.
2. Gently dry up the wound and its surrounding area.
3. Take the dressing with protective transparent foil out of the sterile pouch. Remove the foil from one side of the dressing and place the dressing onto the wound, sterile side with no foil on the wound. When placing it, make sure that the dressing covers the whole wound. It can be cut or fold and it may stretch over the wound margins.
4. Remove the protective foil from the other side of the dressing.
5. Protect the dressing by placing a secondary absorbent dressing of your choice over it. In case of smaller amounts of secretion, nonwoven compresses may be used. For wounds with larger amounts of secretion, use a highly absorbent dressing or polyurethane foam. Fasten securely with a fastening bandage, preventing the dressing from moving.

For easier application of Vivamel – alginatne dressing with medical chestnut honey use sterile medical devices, e.g. sterile tweezers or tongs. The dressing can be cut in smaller pieces with sterile scissors while still protected by the protective transparent foil.

**HOW OFTEN SHOULD ONE CHANGE VIVAMEL – ALGINATE DRESSING WITH MEDICAL CHESTNUT HONEY?**

At the beginning of the healing process, it is recommended that the dressing be changed every two days, in case of substantial amount of secretion even every day, for the effects of honey are decreased by dissolving in wound secretion. Wounds with very little secretion require changing only twice a week. Every time before changing a dressing, remove any remaining alginate or honey from the previous dressing and dispose of it as

infected waste. Clean and dry up the wound and place a new dressing as described above.

**POSSIBLE SIDE EFFECTS**

Due to honey's high osmotic activity, which is based on driving water molecules out of the wound, a burning sensation or a slight pain can occur after applying the dressing on the wound, but it usually goes away after 30 minutes to 2 hours. In case of unbearable pain, stop using Vivamel – alginatne dressing with medical chestnut honey. Should any unknown side effects occur during the use of the product, ask your doctor or pharmacist or consult the manufacturer or distributor.

**WARNINGS**

Do not use the dressing if you are allergic to honey or have already experienced a hypersensitivity reaction to honey in the past. Do not use the dressing if the protective pouch has been damaged or opened. Do not use the dressing if the welds of the packaging are not sealed. Once the alginate dressing has been opened, never sterilise and reuse it. Before and after the application of the dressing, always wash your hands. The dressing is not suitable for eating. If consumed, it can cause blood sugar level to rise in diabetes patients or a hypersensitivity reaction in persons with honey allergy. Avoid contact with eyes and mucosa. In case of accidental contact with eyes or mucosa, rinse with abundant quantities of water. For single use only. Never reuse a used dressing. A used dressing does not contain honey or only a small amount of it, which decreases its effect. Absorbency of a used dressing is also reduced. Reusing a dressing can lead into contamination of the product or the wound, possibly even an infection. One dressing should never to be used on two patients, in order to prevent contamination of the product or the wound, possibly even an infection.

Clinical studies of wound healing with Vivamel medical honey in patients with diabetes show very positive results, however, the treatment must be performed under a physician's supervision. Shelf life of the dressing is three years from the production date. After the expiration date, do not use the dressing and dispose of it as normal waste. After four weeks of therapy with Vivamel alginatne dressing, a doctor should review the treatment.

**STORING VIVAMEL – ALGINATE DRESSING WITH MEDICAL CHESTNUT HONEY**

Store product in a dry place at temperatures between 5 and 35 °C horizontally, otherwise honey will start leaking from the dressing. Avoid exposing the product to direct sunlight.

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